



## TASSO-M20

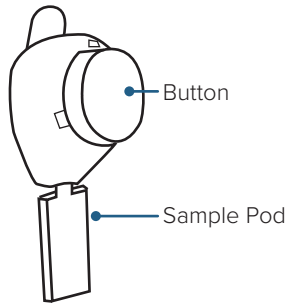

## PREPARE

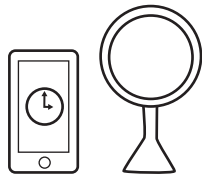

### 1. GET SUPPLIES

You will need a timer and a mirror during collection.



### 2. WASH HANDS

Use soap and warm water.



### 3. WARM ARM


Expose your shoulder and rub quickly just below your shoulder until it feels warm.



### 4. CLEAN ARM


Using the alcohol pad, wipe the warmed area. Let it dry.

## COLLECT

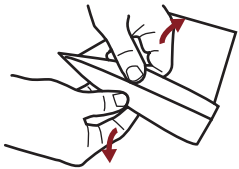

### 5. OPEN WHITE POUCH


Pull the white and clear layers of the pouch apart from the middle to open.

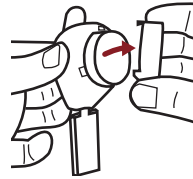

### 6. REMOVE CAP

Pull off the clear plastic cover over the red button.


**DO NOT PRESS THE RED BUTTON YET.**

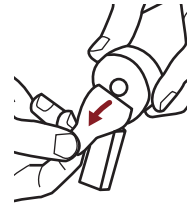

### 7. PEEL BACKING

Pull the tab behind the red button to remove the paper backing.


**KEEP SAMPLE POD POINTING DOWN.**

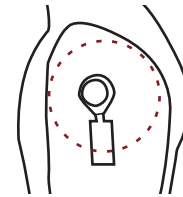

### 8. STICK DEVICE

Stick the device to the warm and clean area on your shoulder.


**DO NOT REMOVE ONCE IT'S ON.**

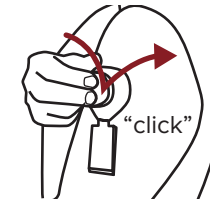

### 9. PRESS BUTTON

Press the red button firmly. Let go when you hear a loud click. You won't see blood right away.

**START A 5 MINUTE TIMER.**

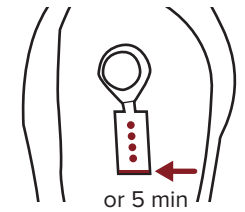

### 10. WAIT FOR BLOOD

Wait 5 minutes. Watch in the mirror and continue to next step before 5 minutes if blood reaches the bottom.

**KEEP ARM AT SIDE DURING COLLECTION.**

## RETURN



### 11. PEEL OFF DEVICE

Slowly peel the device off your arm.



### 12. OPEN VENT


Peel the clear film to expose the sample pod vents.



### 13. PACK FOIL BAG

Place the used device in the silver return bag and seal.

**LEAVE THE MOISTURE PACKS IN THE BAG.**

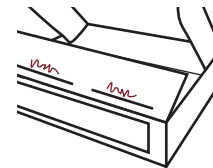

### 14. DATE & TIME

Write your collection date and time on the inside flap of the box.

If needed, apply the bandage to your arm.



### 15. SEAL BOX

Place sample in the box and seal box with the strip.



### 16. MAIL PACKAGE

Before 24 hours, drop the box at a post office or in a blue USPS (United States Postal Service) mailbox.


**INSTRUCTIONS FOR USE**

# TASSO ONDEMAND

## TASSO-M20

Allows self-collection of a single blood sample that is shipped to a lab for analysis by a medical professional.



 Contact us for help:  
1-408-708-8451

**WARNINGS**

Prescription use only.

Use before the expiration date affixed to the box.

One-time use only.

Keep out of reach of children.

Minor bruising may occur around the sample collection site.

For external use only.







Not for use on infant heels.

Intended for self-use or use by a healthcare provider.




Use while seated as fainting may occur with any blood sampling procedure.



Not intended for Newborn Screening or HbA1c testing.

**KIT CONTENTS**

-  Use by date
-  Consult instructions for use
-  This device contains sharps
-  Do not use if package is damaged
-  Single use only
-  Store at 15-30°C

**SYMBOLS**

-  REF Catalog number
-  LOT Lot number
-  IVD For in vitro diagnostic use

-  Rx ONLY Prescription use only
-  STERILE R Sterilized by irradiation

PN0009-01, 04

Tasso, Inc.  
1631 15th Ave. W., Suite 105  
Seattle, WA 98119

